


# Harness Instruction Manual

## Track Mounted Restraint

**You are responsible to tiedown the wheelchair and patient before moving the vehicle.**

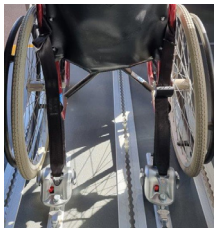

### STEP 1

Place wheelchair central between tie down points on the vehicle floor facing the driving position.




### STEP 2

Slip wheelchair restraints into disc or into tracking.



### STEP 3

Fasten belts to ridged frame on wheelchair in all 4 positions.

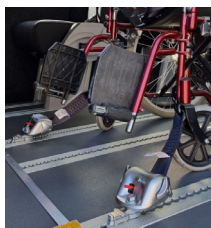

### STEP 4

Attach front anchor point to wheelchair and anchor to the floor. (Never cross over the belts).



### STEP 5

Attach tie down as far apart as possible especially at rear.



### STEP 6

Tighten tie downs to avoid webbing slackness.



### STEP 7

Attach pelvic belt (lap belt) to harness and fit across occupant between armrest and backrests, as tight as possible. Pelvis belt should be as low as possible around patient.

### STEP 8

Attach over shoulder belt across the shoulder right in the centre collar bone. (Optional if supplied).